EN - MONOPROTECT INIBSA

Needsles for dental use with threaded hub for cartridge syringes, sterile, silicone-coated, disposable, non-toxic and non-pyrogenic when in unopened and undamaged package. The welding point joining the cap to the protector ensures that the sterility of the needle is maintained.

INDICATIONS

These needles should only be used by dentists to perform local dental anaesthesia.

INSTRUCTIONS FOR USE

Positioning the needle on the syringe.

- 1) Break the guarantee seal.
- 2) Pull while twisting to remove the cap.
- 3) Screw the threaded base onto the syringe, taking care not to remove the protector.
- 4) Remove the protector with a slight twisting motion just before anaesthesia.

After use:

- 1) Carefully remove the needle from the syringe to avoid accidental puncture.
- 2) Dispose of the used needle immediately in a suitable container conform to regulations, without bending it, break it or handle it in any other way.

WARNINGS

- Do not use if the guarantee seal is not intact.
- Do not reuse. Should the needle be accidentally reused, there is a risk of contamination through cross infection.
- Do not recap the needle by hand, as this may cause accidental puncture.
- Do not bend the needle or apply excessive pressure on it, as serious injury to you and/or your patient may occur.
- If any serious incident has occurred in relation to the device, the user should be report it to the manufacturer and the competent authority of the Member State in which the user is established

CAUTIONS

Keep in original packaging until use. Store at room temperature and protect from moisture. Only for professional use in the dental field for the administration of anaesthetics

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